


FITTING GUIDE



Instructions For Use

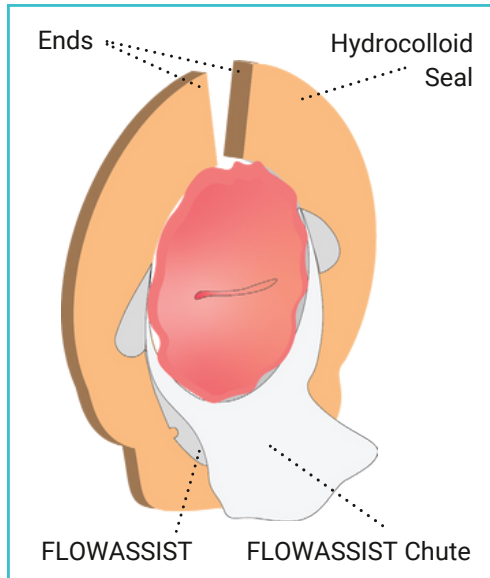

Step 1

Cut the pouch opening to your stoma size plus 5mm in diameter, to accommodate the FLOWASSIST Chute
E.g. for a 25mm stoma, cut the pouch opening to 30mm
All stomas are different so some alterations may be required to achieve the ideal fit

Step 2

- Wash your hands
- Clean and dry the skin around your stoma
- Warm the Seal in your hand for 30 seconds
- Remove the paper liners

Step 3

Position the FLOWASSIST Chute below the stoma or rotate to align with the direction of output.

Step 4

- Wrap the ends of the Hydrocolloid Seal around the stoma ensuring the ends overlap.
- Mold for optimal fit.

5A 1 piece pouch **5B 2 piece pouch**

Step 5

Ensure the FLOWASSIST Chute is fully inside the pouch opening, then pull the pouch up underneath the Chute, as far as it will go.

Step 6

Apply light pressure to the pouch to achieve secure adhesion.

NOTE

Visual Inspection: If possible, check that no part of the Hydrocolloid Seal is visible. You should only see the FLOWASSIST Chute, if you notice that an area of the Hydrocolloid Seal is exposed, try to adjust the pouch to cover this area.

Why just absorb stoma output when you can **direct it away** from the skin?



Protective Non-Absorbent Polymer

- Shields the hydrocolloid minimising absorption of corrosive output
- Prevents hydrocolloid breakdown, so the seal lasts longer
- Prevents leakage and prolongs appliance wear time



Mouldable Hydrocolloid

- Skin-friendly, longer-lasting material
- Can mould and shape for optimal fit
- Split ring enables an accurate, secure fit

Unique Patented FLOWASSIST Technology

- Soft, flexible FLOWASSIST chute channels output away from the skin and into the pouch
- Protects the skin from corrosive output
- Ostoform Seal can be rotated and positioned for the chute to align with individual stoma openings

Pouch Positioning Slot

- Ensures an accurate, secure pouch attachment
- Compatible with one-piece and two-piece pouches
- Compatible with flat and convex pouches

Ostoform **FLOWASSIST**.... an innovative solution for Ostomy Care

Ideal for Patients:

- Suffering from leakage
- Experiencing peristomal skin conditions
- With high output, fluidic stomas
- Requiring extended wear time
- As an alternative to a hard convex pouch
- Where the stoma Os is not centred

CONTACT RHODES PHARMA LTD FOR SAMPLES OR MORE INFORMATION